



Ultra-Soft is a PVC healthcare fabric manufactured using the casting process. This provides a soft and durable finish for mattress covers and other general purpose applications. Ultra-Soft includes all the key elements necessary for a healthcare environment making it a versatile performer.

Product Features

- Soft Handle
- Fluid Proof
- Fire Retardant
- Hypoallergenic / Non-irritating
- Anti-bacterial
- Easy to Maintain
- Anti-static
- Stain Resistant

Typical Applications

- Healthcare Mattress Covers
- Childcare Mattresses
- Support Cushions
- Heavy Duty Pillow Covers
- Hostels & Dormitories





Product Specifications

Roll Width: 137cm

Standard Roll Length: 50m

Property	Standard/Test Method	Result
Total Weight	AS 1441.2	510g/m ²
Tear Strength	AS 2001.2.10	30N min (warp) 20N min (weft)
Tensile Strength	AS 2001.2.3	300N min (warp) 250N min (weft)
Adhesion	AS 1441.5	25N/5cm min
Flame Resistance	AS 1441.13	5 sec max
Hydrostatic Head	AS 220.2.17	>100kpa
Anti-microbial	AS 1157	No fungus growth
Electrical Resistivity	BS 3837	7.6 x 10 ¹⁰ max



Care & Maintenance

Healthcare Fabrics require regular care & maintenance. If the recommended instructions are followed, the end user can expect to maximise the fabrics potential life and maintain a hygienic environment.

General Cleaning

Regularly clean **Ultra-Soft** Healthcare Fabrics with a mild solution of phenolic based cleaner and water. Rinse thoroughly with fresh water and wipe dry.

Spot Cleaning

For difficult spills and stains, prepare a strong solution of phenolic based commercial cleaner. Using a soft bristle brush and solution, clean the affected area. Rinse thoroughly with fresh water and wipe dry.

Disinfection

It is common to disinfect surfaces in a healthcare environment. Using a water/bleach solution of 10:1, clean the entire mattress. The bleach must not have a sodium hypochlorite level in excess of 5%. Disinfectants applied at full concentration will decrease the life of **Ultra-Soft** Healthcare Fabrics. Rinse thoroughly with fresh water and wipe dry.

Notes

- Never exceed the cleaning manufacturers concentration recommendations.
- Do not use abrasive cloths or cleaners.
- Do not use aggressive cleaners or chemicals.
- **Ultra-Soft** Healthcare Fabrics are not autoclavable.
- When in doubt, contact your local Nolan.UDA branch for further advice.